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Brevet européen n° 1439866, 1409034

*Gold MH et al. Dermatol Surg 2014;40: 825-831

KELO-COTE® Gel is a lightweight, transparent, self-drying silicone gel for the treatment of scars. **KELO-COTE® Gel** rapidly dries to form a flexible, waterproof sheet which forms a protective barrier against chemical, physical and microbial invasion of the scar site while assisting with hydration. This creates an environment which allows the scar to mature through normalised collagen synthesis and helps improve the physiological and cosmetic appearance of the scar.

KELO-COTE® Gel has been shown to flatten, soften and smooth scars, relieve the itching, discomfort and pain of scars, as well as reduce the discolouration associated with scars. Silicone gels have been shown to be effective in treating abnormal scarring in the form of hypertrophic scars and keloids. The 2014 International Clinical Recommendations on Scar Management supports the use of silicones as the preferred first line therapy for the treatment of linear hypertrophic scars, minor keloids and in the prevention of hypertrophic scars.*

Intended use:

KELO-COTE® Gel is for the treatment and prevention of hypertrophic and keloid scars resulting from burns, general surgical procedures and trauma wounds.

KELO-COTE® Gel is suitable for use on the face and body. **KELO-COTE® Gel** can be used on joints, flexures, hairy areas and large area scars such as those caused by burns or major trauma. **KELO-COTE® Gel** is suitable for use on children over the age of 3 months and people with sensitive skin.

Directions for use:

Ensure that the affected area is clean and dry. If using the 60g tube, remove cap and peel off foil before application. For the 6g and 15g tube, cut the tip of the nozzle with clean scissors before application. Apply **KELO-COTE® Gel** to the area using a clean finger twice daily as a very thin layer and allow to dry. For maximum effect, **KELO-COTE® Gel** should have 24 hour contact with the skin. Once dry, **KELO-COTE® Gel** can be covered with sun block or cosmetics. Replace cap after use.

If **KELO-COTE® Gel** has not dried within 4-5 minutes, you have probably used too much. Gently remove the excess and allow the drying process to continue.

Recommended minimum treatment time is 60-90 days. Larger and older scars may take longer and continued use is recommended if improvement is still seen after the initial 90 days.

Caution:

For external use only. Do not use if you are hypersensitive to any of the ingredients in **KELO-COTE® Gel**. Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. **KELO-COTE® Gel** should not be used on dermatological conditions that disrupt the integrity of the skin. If irritation occurs, discontinue use and consult your physician.

KELO-COTE® Gel may stain clothing if it is not completely dry. Keep away from sources of ignition and do not smoke while applying the product or while the product is drying. Keep out of the reach of children. Store below 25°C. Do not use after the expiration (EXP) date printed on the tube.

Ingredients: Cyclopentasiloxane, Dimethicone, Silica, Dimethicone Crosspolymer

Contents: 6g, 15g, 30g & 60g

Lot number and Expiry date are printed on the base of the carton.

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Advanced Bio-Technologies, Inc, 4830 West Kennedy Blvd, Suite 600, Tampa, Florida 33609, USA
Alliance Pharma S.r.l., Viale Restelli 5, 20124 Milano, Italien